DE

selflab

SCHNELLDIAGNOSETESTS

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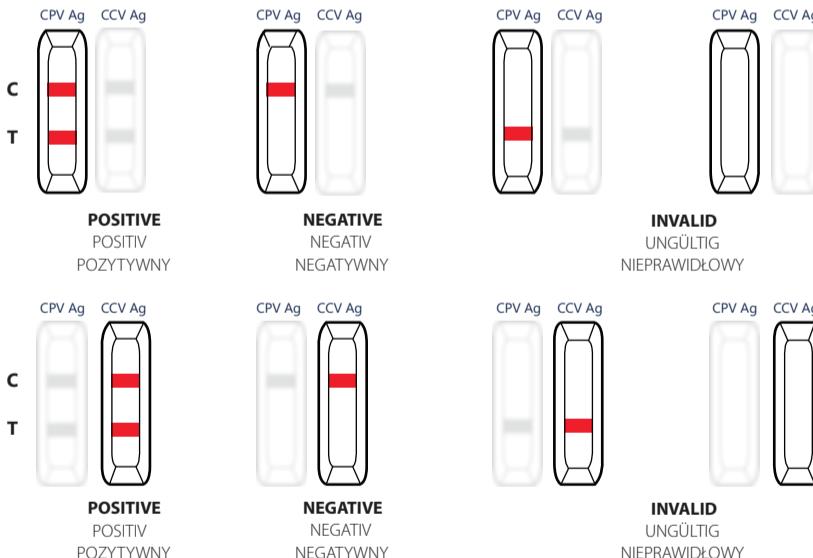
INTERPRETATION OF RESULTS

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EN Positive result: there will be 2 lines, 1 in control field (C) and 1 in the test field (T). **Look closely!** The presence of any line in test field (T), even very vague, means that the result is considered positive.

Negative result: only 1 clear line in control field (C), no line in test field (T).

Invalid result: no color line in control field (C), the invalid result is not affected by the appearance of the test field (T) (presence or absence of a line in this field).

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EN Test principle:

The test is performed by a lateral flow immunochromatographic method (to detect the presence or absence of the target Ag in the sample). The test cassette has a window (S) into which the sample is dropped and two fields: test field (T) and control field (C). When the sample is placed in the (S) window, the liquid will flow laterally into test field (T). If the antigen is present in the sample, a visible (T) line will appear.

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selflab

RAPID DIAGNOSTIC TESTS

Gastrointestinal diseases in dogs
 - parvovirus, coronavirus

The Selflab CPV/CCV Ag test is used to detect parvovirus and coronavirus antigens in dogs feces. Antigen (Ag) is a foreign substance that stimulates the body to produce antibodies.

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Storage of Tests:

Store the test kits in a temperature 2-30°C. Do not freeze, protect from direct sunlight.

Sample Storage:

The collected sample should be tested immediately.

USE CAUTION:

Samples are considered infectious material! Expiration date is indicated on the package. Tests are for animal use only.

Performance:

Test specificity is the ratio of the true negatives to the sum of the true negatives and false positives combined and was: for CPV 100%, CCV 99.2% for this assay. It means that 100% and 99.2% of samples from healthy individuals with this test are marked as healthy / negative.

The sensitivity of the test is the ratio of the true positives to the sum of the true positives and the false negatives. The sensitivity: of CPV 97.6% and CCV 94.9% estimated in this test, means that such a percentage of sick individuals will be positively diagnosed.

LIMITATIONS:

1. The test result should not be the sole source of diagnosis, as with other diagnostic tests, the result should be interpreted by the veterinarian with the other clinical data.
2. False positive result may occur when:
 - there is a cross-reactivity,
 - antigens of another pathogens are present.
3. False negative result may occur when:
 - the specimen has been sampled incorrectly,
 - antigens levels in the sample are too low,
 - the specimen has been sampled too early or too late, the tested antigen is not present in the feces.
4. Due to the possibility of false-positive or false-negative results, cassette test results may be confirmed by other methods.
5. Invalid test result may occur when:
 - the specimen was sampled incorrectly,
 - sediment has been placed in the test well "S",
 - not enough drops of sample material have been placed in the test well "S".
6. If the symptoms persist despite the negative or invalid results, it is recommended to do the test again within 24-48h and contacting your veterinarian.
7. Negative result does not exclude a possibility of infection.
8. Positive result does not exclude a possibility of coexisting infection with different pathogens.
9. Each positive result should be consulted with a veterinarian.

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