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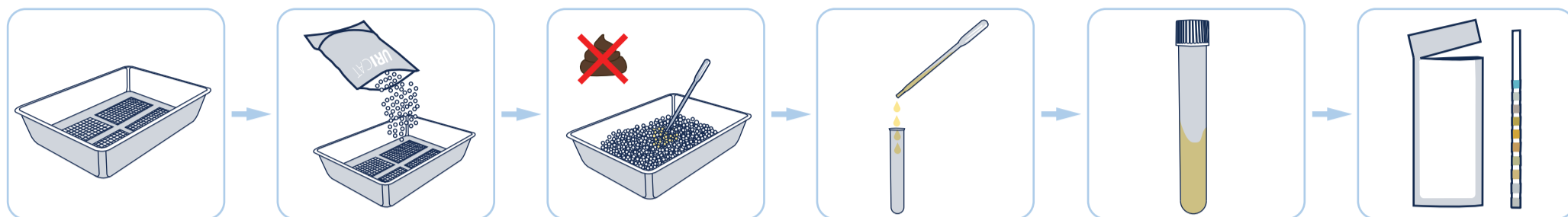
**EN** Wash and rinse your cat's litter box in plenty of water to remove any remaining detergent. Then dry it with a cloth, paper towel or toilet paper.

Put URICAT litter into the prepared litter box.

Remove feces and other substances immediately, as they may contaminate the urine sample. Collect a urine sample as soon as possible using the pipette provided. Squeeze the pipette and take the sample from a tilted litter box.

Put the urine sample into the tube using the pipette.

Close the tube with the cap.

Remove the Selflab Urine Test strip from the pouch.

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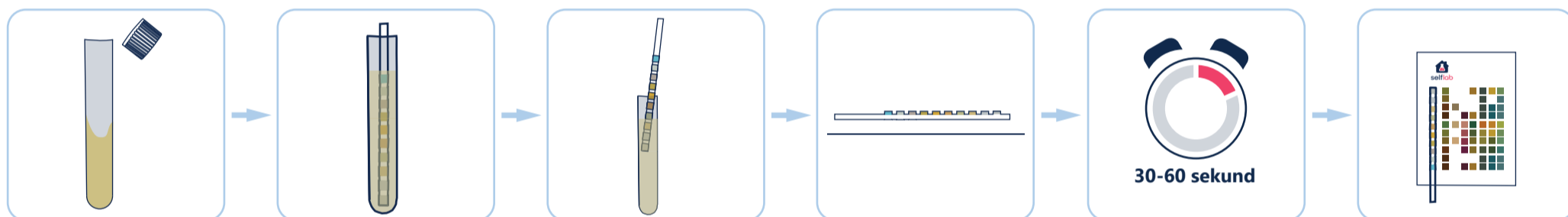
**EN** Take off the cap from the tube.

Immerse all reagent areas in the urine sample for 1-2 seconds.

Remove excess urine from the strip by delicately tapping the edge of the strip on the sample container or gently dry the edges of the strip with a tissue.

Hold the test strip in a horizontal position.

Read the results within 30-60 seconds

Compare the results to the color chart with result interpretation included in the test kit. Color changes that appear only along the edges of the area or after a certain amount of time are not diagnostically significant.

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